

## **SkyJet 2.8 and 3.2 Injector Set**

Handling instructions for preloaded  
CT ASPHINA 603P and CT SPHERIS 203P



We make it visible.

# SkyJet 2.8 and 3.2 Injector Set

Preparing the preloaded CT ASPHINA® 603P and CT SPHERIS® 203P for use



1 Check that the plunger is fully withdrawn and held by the safety clips.



2 Insert the cassette into the injector following the arrows. Push all the way in until you hear a "click".




5 Hold the breakable safety tab between thumb and index finger. Snap it in the clockwise direction. The tab breaks and releases the lens.



7 Introduce viscoelastic into the hole on the back of the injector. Fill the first 3 mm of the injector nozzle up to the line.



8 Gently arm the plunger to release it from its safety clips and slowly push to drive the lens into the injector nozzle.

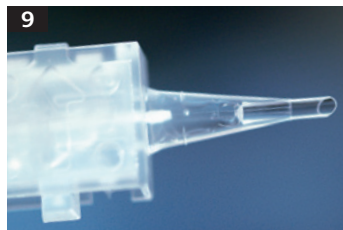

10 With the bevelled orifice facing downwards, insert the tip of the injector into the incision and inject the lens into the capsular bag.



0459 SkyJet  
CT ASPHINA 603P  
CT SPHERIS 203P



**Carl Zeiss Meditec AG**  
Goeschwitzer Str. 51–52  
07745 Jena  
Germany  
[www.meditec.zeiss.com/iol](http://www.meditec.zeiss.com/iol)  
[www.meditec.zeiss.com/contacts](http://www.meditec.zeiss.com/contacts)



**Carl Zeiss Meditec SAS**  
27 Avenue Paul Langevin  
17180 Périgny – France  
**Mailing address:**  
Avenue Paul Langevin – BP5  
17053 La Rochelle – Cedex 9  
France

EN\_32\_022\_00271  
CT ASPHINA, CT SPHERIS and SkyJet are registered trademarks of Carl Zeiss Meditec. The contents of the brochure may differ from the current status of approval of the product in your country. Please contact our regional representative for more information. Subject to change in design and scope of delivery and as a result of ongoing technical development. Printed on elemental chlorine-free bleached paper.  
© 2015 by Carl Zeiss Meditec AG. All copyrights reserved.

